



SUGGESTED INSTRUCTIONS FOR USE

REUSABLE BRUSHES (except Channel brushes)

(ITEM NO.)	DESCRIPTION	MATERIAL	UOM
20-XXXXX 22-XXXXX 23-XXXXX 24-XXXXX 25-XXXXX 26-XXXXX	All reusable IN-BRUSH™ Twisted Wire Brushes	Stainless Steel	3/BAG
28-69394 29-69394 28-72550 29-72550	In-Brush™ Plastic Handle Brushes	Polypropylene	3/BAG
36-20010	Reusable External Cleaning Brush	Polypropylene	3/BAG
40-18511	Reusable Valve/Control Cleaning Brush	Delrin	3/BAG
40-XXXXX 41-XXXXX 411-XXXXX	All reusable Arc™ STEAMABLE Brushes	Delrin	3/BAG
50-ACETAB	Reusable Acetabular Reamer Cleaning Brush	Delrin	3/BAG
50-10295	Reusable Jar/Bottle Brush	Delrin	3/BAG
50-4XXXX	Reusable Flat Cleaning Brushes(Instrument Cleaning Brushes)	Delrin	3/BAG
50-40800	Reusable Precision External Cleaning Brush	Delrin	3/BAG
50-40911	Reusable Hand scrub brush	Polypropylene	3/BAG

DESCRIPTION/INTENDED USE

Batrik's reusable brushes are intended to assist in cleaning of medical devices in conjunction with a suitable cleaning solution.

All components used in the manufacturing of our reusable brushes are made of commonly used materials and do not require special sterilization handling outside your facility's standard sterilization protocol. At your discretion, please feel free to consult with your sterilization equipment supplier as they should have a list of acceptable serializable components/materials.

INTENDED PURPOSE

Brushes are a medical device used to clean endoscopic devices and/or other instruments used in the healthcare industry.

INTENDED USERS

Healthcare professionals.

INDICATIONS

Brushes are a medical device used clean endoscopic devices and/or other instruments used in the healthcare industry.

CONTRAINDICATIONS

Do not use these brushes for any purpose other than their intended use. Patient injury, cross-contamination and equipment damage may occur.

WARNING/PRECAUTIONS

- ⚠ Consult the manufacturer's IFU for the device you are cleaning and follow those instructions closely.
Pay close attention to recommendations on the material composition of the brush trim (nylon, stainless-steel).
- ⚠ Brushes are intended for use as cleaning tools only. They are not for use in patient care or aseptic techniques.
- ⚠ Avoid brush compression during sterilization and storage.

INSTRUCTIONS FOR USE

1. Pick the correct type of cleaning brush for the cleaning process.
2. Between usages, inspect for wear, fraying or damaged bristles. If observed, the brush should be discarded in compliance with your facility disposal instructions.
3. Cleaning and Sterilization: These products have been manufactured using materials that withstand repeated steam sterilization cycles achieving 134 °C (273 °F). Although the product is designed for considerable re-use, variables such as frequency of sterilization, cleaning application and handling can impact the product’s useful life. When cleaning the brush ensure it is free from all foreign material and thoroughly cleaned before being processed at sterilization temperatures. Following sterilization or any established cleaning protocol, examine the brush for any loose bristles, overly worn bristles or cracks on the sheaths.

STORAGE

Store in a manner that will reduce cross-contamination. Keep in cool dry place and away from direct sunlight.

DISPOSAL

After use, this product may be a potential biohazard. Handle and dispose of the product in accordance to facility protocol with accepted medical practice.

NOTICE TO USERS AND/OR PATIENTS

Any serious incident that has occurred in relation to the device, should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

LABEL SYMBOLS



Keep in dry area



Keep away from direct sunlight



Non-Sterile product



Does not contain natural latex



CE Marked



UKCA Mark



Indicates the item is a medical device



Manufacturing date



Caution, consult accompanying documents



Unique Device Identifier

BATRIK MEDICAL MANUFACTURING INC.
2165 46th Ave., Lachine, QC, H8T 2P1 (CANADA)
Tel. +1(514) 631-7988 Fax. +1(514) 631-9083
www.batrik.com

Advena Limited.
EC REP Tower Business Centre,
2nd Flr., Tower Street,
Swatar, BKR 4013 Malta

UK Responsible Person
Advena Ltd UK
Pure Offices
Plato Close
Warwick, England CV34 6WE

CH REP Swiss AR Services GmbH
Industriestrasse 47
CH-6300 Zug

Issue date: 11/19/2019
Revision date: 10/02/2023

