

INSTALLING THE POLYP TRAP

Connection:

1. Connect tubing fitting end of trap base to endoscope suction adaptor.
2. Connect suction connector on trap base to suction system.

Capture:


Apply suction via the endoscope suction valve to trap polyp.

Polyp Removal and Transport:

3. After the polyp is trapped, remove the drawer chamber from the endoscope and suction system.
4. Separate trap by pull out the drawer chamber.
5. Insert another drawer filter chamber in to trap base for more polyp trap.
6. Remove polyp and place in specimen jar with fixative (if possible use the hollow stick help to remove the polyp), or place trap in specimen jar and fill with fixative.
7. Transport to pathology.

Product Disposal:

After use, this product may be a potential biohazard. Handle and dispose of the product in accordance with accepted medical practice and applicable local, state and federal laws and regulations.

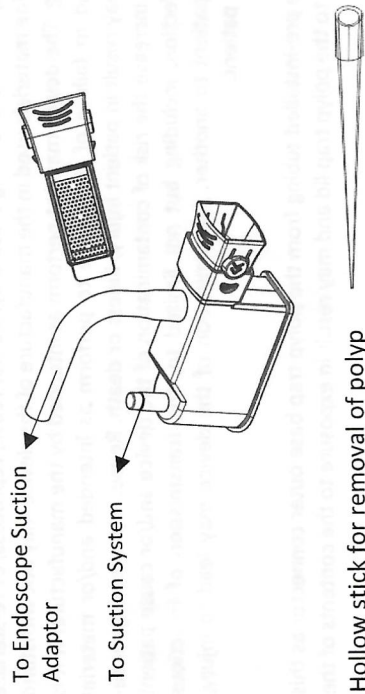

DESCRIPTION OF SYMBOLS USED ON LABELS

	Non-Sterile Device		Instruction for Use
	Do Not Re-use		Quantity in package
	Manufacturing Date		Manufacturer
	Reference No.		Lot Number
	Use by		Keep Dry
	Authorized Representative in the European Community		Prevent from sun light
	Latex Free		DEHP Free
	EC Conformity		Temperature limit
	Humidity Limit		Medical Device
	Do not use if package is damaged or open		

Polyp Trap With 2 Drawers

Instructions for Use.

Read carefully prior to use.

Suzhou GZM Medical Co., Ltd.

Room1612, Building 8, Shi Shan Tian Jie Living Plaza,

215011, Suzhou, Jiangsu,

PEOPLE'S REPUBLIC OF CHINA

Tel +86-512-65575235

e-mail: sales@gzmmedical.com

www.gzmmedical.com

Made in China

Reference Number

G-PLP02

Description

Polyp trap with 2 drawers

FR Piège à polypes à 2 tiroirs / DE Polypenfalle mit 2 Schubladen/ES Trampa para pólipos de 2 cajones/NL Poliepenval met 2 lades / PT 2 gaveta para pólipos armadilha/ IT Trappola per polipi a 2 cassetti/ DK 2 skuffepolypapper fælde/ GR Πάγιδες 2 πολυπόδων συρταριών/ SE 2 lådpolyperlås / HU 2 fiók polip csapda/ PL Pułapka na polipy z 2 szufladami / EE 2 sahtlipoliüüpi / HR 2 ladice za polipe/ CZ 2 zásuvky na polypy / LT 2 stalčių polipai/ SI 2 predal za polipe

INTENDED USE

The single use Polyp Trap is used for suction retrieval and transportation of endoscopically removed polyps for histological examination. The Polyp Trap has two drawer chambers to collect the polyps.

The single use two chamber polyp trap consists of:

- Two drawer chambers with filtered trap to collect polyps alternatively inserted into trap base.
- A trap base have two connectors for connections to endoscopes and suction source.
- A straight tubing on connector for connection to the endoscope.
- A polyp removal hollow stick

WARNINGS AND PRECAUTIONS

Products marked “single-use” are for single-use only. Do not reuse, reprocess, or re-sterilize single-use products. The materials used in the manufacture of the device may not withstand repeated reprocessing. The device may not perform as intended by the manufacturer if it is reused. This may lead to failure of the device to perform as intended and/or material degradation, which may result in patient injury, illness or death. Reuse, reprocessing, or re-sterilization may also increase the risk of contamination of the device and/or cause patient infection or cross-infection, including but not limited to, the transmission of infectious disease(s) from one patient to another. Contamination of the device may lead to injury, illness or death of the patient.

Do not disconnect the pre-installed tubing from the polyp trap base outer connector as this may result in damage to the polyp trap lid and may result in exposure to the contents of the trap during removal.

Protective garments (i.e. gloves, protective eyewear, gowns, etc) should be worn during the use of this device to protect against the risk of cross-contamination and injury.